

Kit Components

All components are shipped sterile in package.  
The Recibio, Inc. CeaLogic™ Ratio Applicator Kit consists of the following components:

- One (1) Sterile Peel Pack containing all components
- One (1) 12-ml syringe
- One (1) 1-ml syringe
- One (1) Dispenser
- One (1) Dual Lock Adapter
- One (1) Spray Tip
- One (1) Specimen Container
- One (1) Specimen Container Lid




1860 FM 359 #171  
Richmond, TX 77406 USA  
www.recibio.us

Toll free USA 1-844-732-4246

Made in the U.S.A. including foreign components  
Assembled by **Busse Hospital Disposables**

© Recibio, Inc. 2016 All rights reserved.



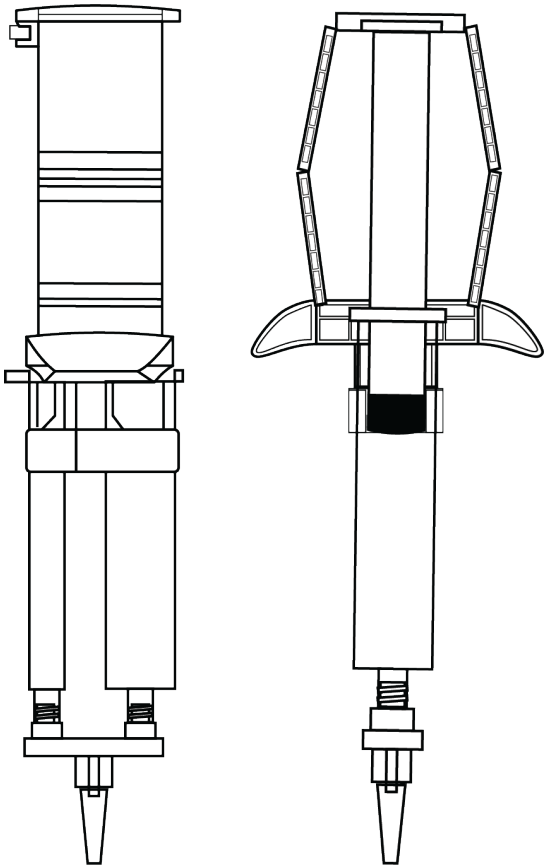






Not made with natural rubber latex



Final Assembly Pictured  
Rev. 4 (08/2019)

Instructions For Use

Explanation of symbols on package labeling

	Caution, Consult Accompanying Documents
	Lot Number
	Catalog Number
	Sterilized Using Ethylene Oxide
	Do Not Reuse
	Use By
	Open Here
	Fragile, Handle With Care
	This Way Up

INTRODUCTION

The **Recibio, Inc. CeaLogic™ Ratio Applicator Kit** consists of **SINGLE USE** components to be used solely for the assembly of the **CeaLogic™ Ratio Applicator**.

**Indications for Use**  
The CeaLogic Ratio Applicator Kit is indicated for use in applying two fluids to a treatment site.

**Description**  
The **Recibio, Inc. CeaLogic™ Ratio Applicator Kit** is intended for the application of fluids as deemed necessary by the surgeon's determination of the clinical use requirements to facilitate the preparation of the treatment site.

**Contraindications**  
The CeaLogic ratio Applicator is contra-indicated for intravascular use.

**Warnings**  
When using two dissimilar fluids, compatibility, appropriateness, and possible interaction must be evaluated. Refer to the surgical fluid, anti-biotic, or other fluid Instructions for Use provided with the fluid to be applied.

**Dosage and Method**  
Dosage is dependent upon procedure and surgeon's determination. To obtain even coverage, keep constant pressure on the dispenser and spray with a sweeping motion.

Cautions

NON-PYROGENIC DEVICE

FEDERAL LAW (USA) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN.

DO NOT RE-STERILIZE.

## Pre-Assembly Instructions

- Using aseptic technique, remove components of the CeaLogic™ Ratio Applicator Kit from the sterile package.
- Fill the larger 12-mL syringe with desired fluid.
- Fill the smaller 1-mL syringe with the second fluid at approximately the same level. The proportions of fluid in the two syringes should be comparable.

## Final Assembly Instructions

- ① Click the 12-mL syringe into the larger side of the Dispenser. The syringe should snap into the Dispenser at both the plunger and the waist.
- ② Click the 1-mL syringe into the smaller side of the Dispenser. The syringe should snap into the Dispenser at both the plunger and the waist.
- ③ Connect Spray Tip to Luer-Lock Adapter and tighten.
- ④ Connect Spray Tip/Luer-Lock Adapter Assembly to the 12ml Syringe and 1ml Syringe and tighten, alternating until complete.

**NOTE:** Depress Dispenser to apply fluids in combination onto desired site.

## Ratio Applicator Assembly

